

INSTRUCTIONS FOR USING THE ZEGLUXEN PEN

Please read these instructions carefully before using your pen.

Your pen comes with 18 mg of liraglutide. You can select doses of 0.6 mg, 1.2 mg and 1.8 mg.

The pen is designed to be used with disposable needles.


Needles are not included in the pack.

BD Ultra fine™ disposable needles or NovoFine® disposable needles are recommended for use with this device.

Prepare your pen

Check the name and coloured label of your pen to make sure that it contains Zegluxen (liraglutide 6 mg/ml). Using the wrong medicine could cause severe harm.

Pull off the pen cap.

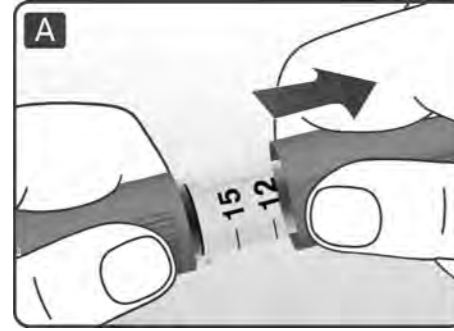


Pull off the paper tab from a new disposable needle. Screw the needle straight and tightly onto your pen.



Pull off the outer needle cap and keep it for later.



Pull off the inner needle cap and dispose of it.

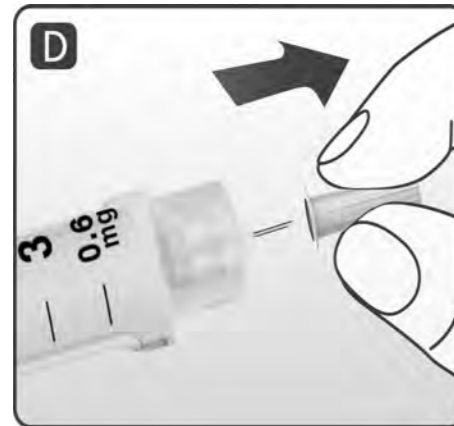

- ⚠ Always use a new needle for each injection. This reduces the risk of contamination, infection, leakage of Zegluxen, blocked needles and inaccurate dosing.
- ⚠ Be careful not to bend or damage the needle.
- ⚠ Never try to put the inner needle cap back on the needle. You may stick yourself with the needle.


Caring for your pen

- Do not try to repair your pen or pull it apart.
- Keep your pen away from dust, dirt and all kinds of liquids.
- Clean the pen with a cloth moistened with a mild detergent.
- Do not try to wash, soak or lubricate it – this can harm the pen.

⚠ Important information

Do not share your pen or needles with anyone else. Keep your pen out of the reach of others, especially children. Change the place where you inject daily to reduce the risk of developing lumps.


Liraglutide Injection Pen

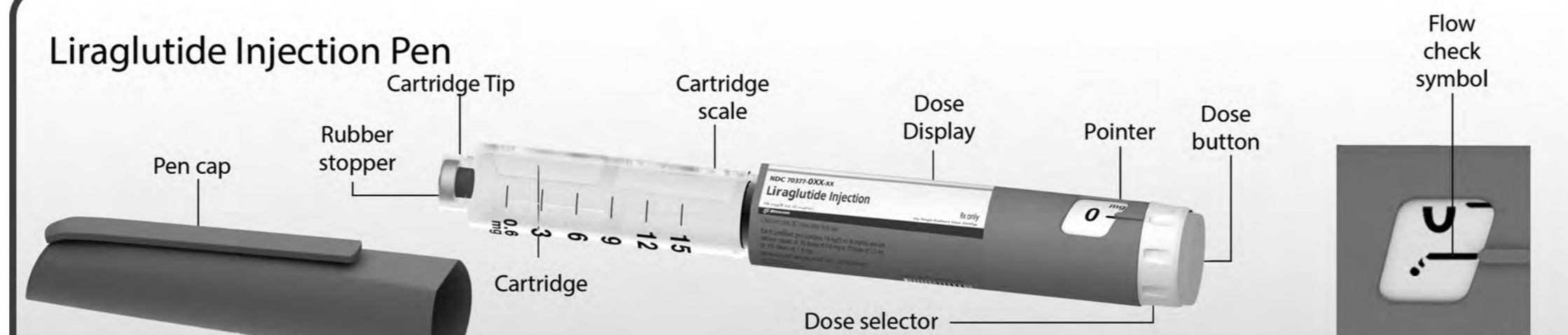


Check the flow with each new needle before injecting.

A new needle should be used with every injection to help prevent infections and blocked needles.

Turn the dose selector until the flow check symbol lines up with the pointer.

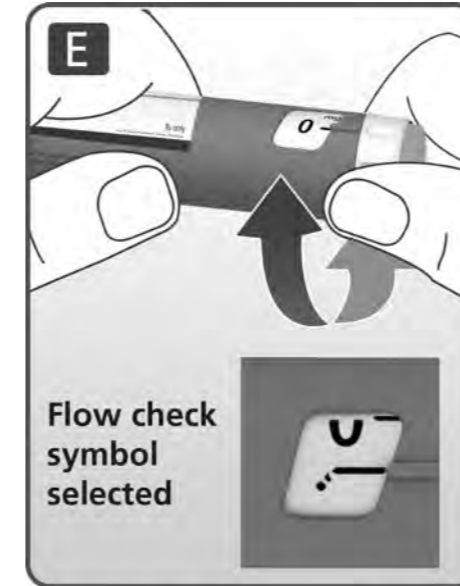


Hold the pen with the needle pointing up. Tap the cartridge gently with your finger a few times. This will make any air bubbles collect at the top of the cartridge.

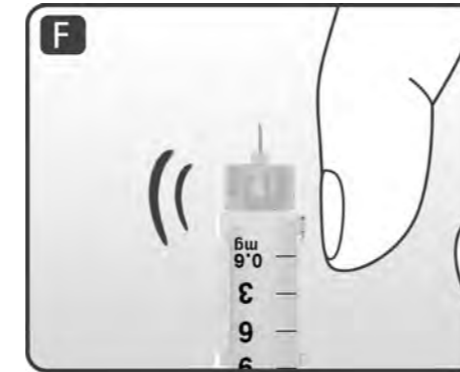

Keep the needle pointing up and press the dose button until 0 mg lines up with the pointer. A drop of Zegluxen should appear at the needle tip. If no drop appears, repeat steps E to G up to four times.

If there is still no drop of Zegluxen, change the needle and repeat steps E to G once more. Do not use the pen if a drop of Zegluxen still does not appear.

This indicates the pen is defective and you must use a new one.

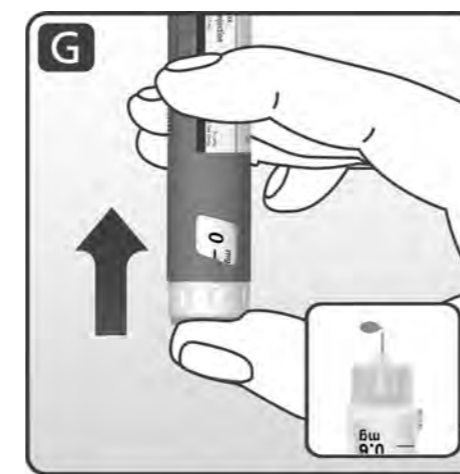

- ⚠ If you have dropped your pen against a hard surface or suspect that something is wrong with it, always put on a new disposable needle and check the flow before you inject.


Select your dose

Always check that the pointer lines up with 0 mg.

Turn the dose selector until your needed dose lines up with the pointer (0.6 mg, 1.2 mg or 1.8 mg).

If you selected a wrong dose by mistake, simply change it by turning the dose selector backwards or forwards until the right dose lines up with the pointer.

Be careful not to press the dose button when turning the dose selector backwards, as Zegluxen may come out.

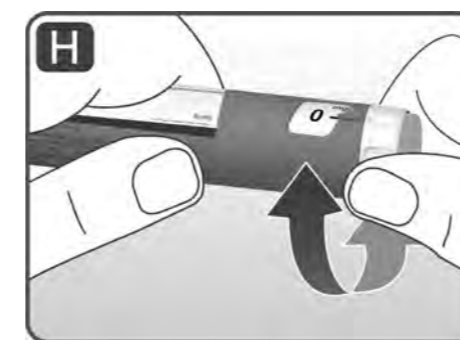

If the dose selector stops before your needed dose lines up with the pointer, there is not enough Zegluxen left for a full dose. Then you can either:

Split your dose into two injections:

Turn the dose selector in either direction until 0.6 mg or 1.2 mg lines up with the pointer. Inject the dose. Then prepare a new pen for injection and inject the remaining number of mg to complete your dose.

You may only split your dose between your current pen and a new pen if trained or advised by your healthcare professional. Use a calculator to plan the doses. If you split the dose wrong, you may inject too much or too little Zegluxen.

Inject the full dose with a new pen:

If the dose selector stops before 0.6 mg lines up with the pointer, prepare a new pen and inject the full dose with the new pen.

0.6 0.6 mg selected

1.2 1.2 mg selected


1.8 1.8 mg selected

- ⚠ Do not try to select other doses than 0.6 mg, 1.2 mg or 1.8 mg. The numbers in the display must line up precisely with the pointer to ensure that you get the correct dose. The dose selector clicks when you turn it. Do not use these clicks to select your dose. Do not use the cartridge scale to measure how much liraglutide to inject – it is not accurate enough.


Inject your dose

Insert the needle into your skin using the injection technique shown by your doctor or nurse. Then follow the instructions below:

Press the dose button to inject until 0 mg lines up with the pointer. Be careful not to touch the display with your other fingers or press the dose selector sideways when you inject. This is because it may block the injection. Keep the dose button pressed down and leave the needle under the skin for at least 6 seconds. This is to make sure that you get your full dose.

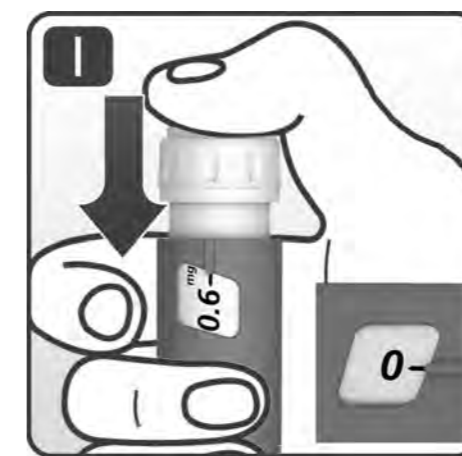


Pull out the needle. After that, you may see a drop Zegluxen at the needle tip. This is normal and does not affect your dose.

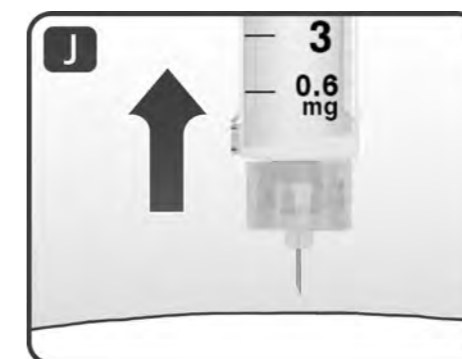


Guide the needle tip into the outer needle cap without touching the needle or the outer needle cap.



Needle (example)

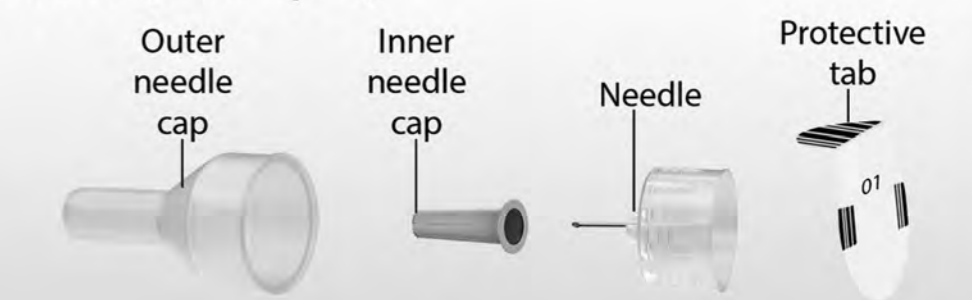

When the needle is covered, carefully push the outer needle cap completely on. Then unscrew the needle. Dispose of it carefully and put the pen cap back on. When the pen is empty, carefully dispose of it without a needle attached. Please dispose of the pen and needle in accordance with local requirements.

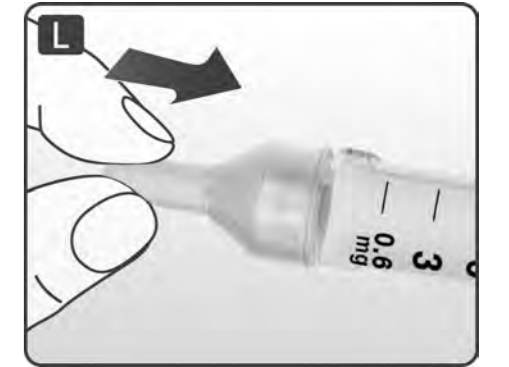

- ⚠ Always remove the needle after each injection, and store your pen without a needle attached.
- ⚠ This reduces the risk of contamination, infection, leakage of Zegluxen, blocked needles and inaccurate dosing.
- ⚠ Caregivers must be very careful when handling used needles – to prevent needle injury and cross-infection.

Last updated July 2025.

Laetus code