

INSTRUCTIONS FOR USE
Dawnzera 80 mg solution for injection in pre-filled pen

This Instructions for Use contains information on how to inject Dawnzera using the pre-filled pen.




Read this 'Instructions for Use' before you start using your Dawnzera pre-filled pen and each time you get a refill. There may be new information. This information does not take the place of talking to your healthcare provider about your medical condition or treatment. Your healthcare provider should show you or your caregiver how to use the Dawnzera pre-filled pen the right way. If you or your caregiver have any questions, talk to your healthcare provider.


- Important information:**
- Dawnzera is injected under the skin (subcutaneous use) only.
 - Each pre-filled pen contains 1 single-dose and can only be used 1 time.
 - Do not remove the clear cap until you are ready to inject Dawnzera (See Step 5).
 - Do not share your pre-filled pen with anyone.
 - Do not use the pre-filled pen if it appears damaged.
- Storage information:**
- Store the pre-filled pen in the refrigerator between 2 °C - 8 °C in the original carton.
 - If needed, the pre-filled pen can be stored at room temperature up to 30 °C in the original carton for up to 6 weeks.
 - If you store Dawnzera at room temperature, write the discard date on the original carton. The discard date is maximally 6 weeks after you take the medicine out of the refrigerator, and should be noted in the space indicated on the original carton for storage at room temperature up to 30 °C.
 - If you do not use the pre-filled pen kept at room temperature within 6 weeks, throw it away.
 - Keep the pre-filled pen in the carton until ready to use.
 - Do not store the pre-filled pen with the clear cap removed.

Keep Dawnzera out of the sight and reach of children.

Dawnzera overview
Single-dose pre-filled pen

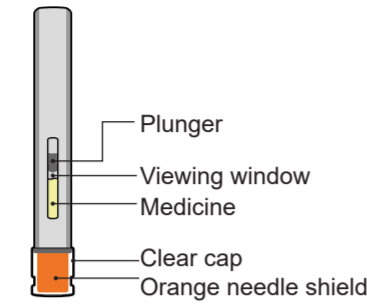


Other supplies (not included)

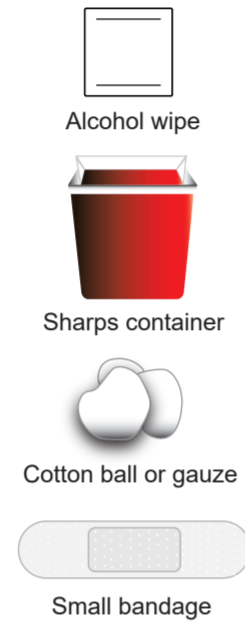

Preparing to inject Dawnzera


Step 1 Remove from the refrigerator

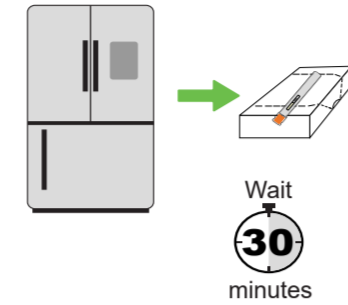

- Remove the pre-filled pen from the refrigerator.
- Keep the pre-filled pen in the original carton and let the pre-filled pen come to room temperature for 30 minutes before injecting.
- When one pre-filled pen from a pack with 3 pens is removed from refrigeration, return the remaining pre-filled pens to the refrigerator for future use until needed.

Do not try to speed up the warming process using other heat sources, such as a microwave or hot water.

Step 2 Inspect the medicine

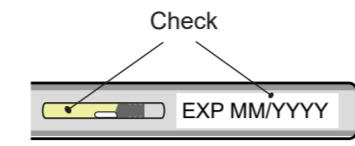


- Check the expiry date (EXP) on the pen and also the discard date on the original carton if the medicine was stored at room temperature up to 30 °C.
- Check the medicine through the viewing window. The Dawnzera medicine should be clear and colourless to yellow. There should be no particles. It is normal to see air bubbles in the solution.

Do not use the pre-filled pen if the:

- clear cap is missing or not attached.
- expiry date (EXP) or discard date has passed.
- medicine looks frozen, cloudy, discoloured, or has particles.
- pre-filled pen appears damaged.

Preparing to inject Dawnzera

Step 4 Wash hands and clean the injection site

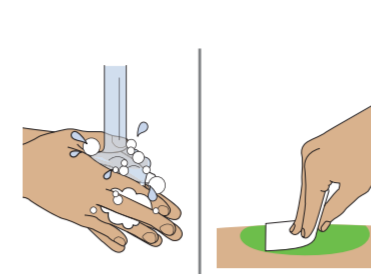


- Wash your hands with soap and water.
- Clean the injection site with an alcohol wipe in a circular motion. Let the skin air dry.

Do not touch the cleaned skin before injecting.

Injecting Dawnzera


Step 5 Remove and throw away the clear cap

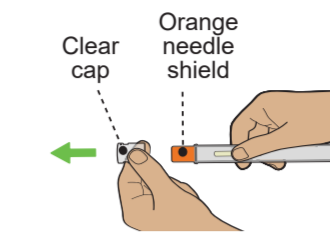

- Hold the pre-filled pen by the middle with the clear cap facing away from you.
- Remove the clear cap by pulling it straight off. Do not twist it off. The needle is inside the orange needle shield.
- Throw away the clear cap in the trash or sharps container.

Do not remove the clear cap until right before you inject. Do not recap the pre-filled pen. Do not push the orange needle shield against the hand or finger.


Step 6 Begin injection

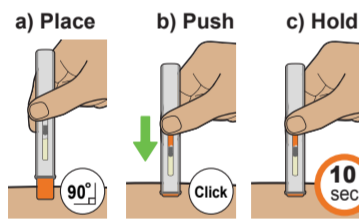

- Hold the pre-filled pen in 1 hand. Place the orange needle shield at a 90-degree angle against your skin. Make sure you can see the viewing window.
- Push firmly and hold the pre-filled pen straight against the skin. You will hear a click as the injection starts.
- Hold the pre-filled pen against the skin for 10 seconds to make sure the full dose has been given.

Do not move, turn, or change the angle of the pre-filled pen during the injection.

Step 7 Finish injection

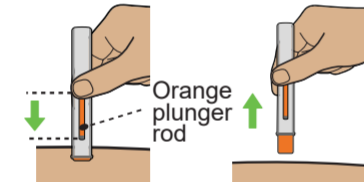

- Check that the orange plunger rod has moved down to fill the entire viewing window. If the orange plunger rod does not fill the viewing window, you may not have received the full dose. If this happens or if you have other concerns, contact your healthcare provider.
- Remove the pre-filled pen by lifting it straight up. After removal from the skin, the orange needle shield locks into place and covers the needle.
- There may be a small amount of blood or liquid where you injected. This is normal.

If needed, press a cotton ball or gauze on the area and apply a small bandage.

Do not reuse the pre-filled pen.

Throwing away Dawnzera

Step 8 Throw away pre-filled pen

- Put the used pre-filled pen in a sharps container right away after use.



Do not throw away the pre-filled pen in your household trash. Do not recycle your used sharps disposal container. Do not reuse the pre-filled pen or clear cap.


If you do not have a sharps container, you may use a household container that is:

- made of a heavy-duty plastic,
- can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
- upright and stable during use,
- leak-resistant, and
- properly labelled to warn of hazardous waste inside the container.

When your sharps disposal container is almost full, you will need to follow your local guidelines for the right way to dispose of your sharps disposal container. There may be local laws about how you should throw away used pre-filled pens.

Do not throw away your used sharps disposal container in your household trash unless your local guidelines permit this. Do not recycle your used sharps disposal container.

Step 3 Choose the injection site

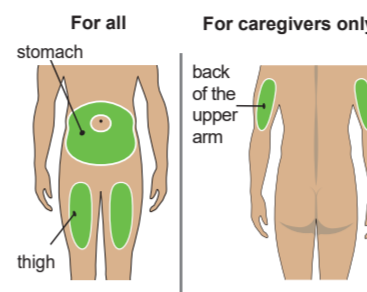

- Choose an injection site on the stomach or the front of the thigh.
- Only caregivers may choose the back of upper arm.

Do not inject:

- within 5 cm of the belly button (navel).
- into skin that is tender, bruised, red, hard, infected or discoloured.
- into scars or damaged skin.
- into the site of the previous injection

DAWNZERA 80 mg
solution for injection
in pre-filled pen

G-4107-01

PRINTING COLOURS		TECHNICAL COLOURS	
Black	Keyline (Non-Printing)	Keyline (Non-Printing)	Technical Info (Non-Printing)
Cyan			
Magenta			
Yellow			

Customer	Otsuka OPEL
Material No.	G-4107-01
Product Code	N/A
Item Description	Dawnzera 80 mg GDE UK
Component Type	Leaflet
Dimensions	320 x 420 mm
Affiliates	UK
Language	English
Proof By	C-JE
Proof No.	3
MPS Artwork Number	4066207
Pharmacode	
Date	29.04.2026

APPROVAL SIGNATURE

DATE

Warning: We cannot accept responsibility for any errors in this proof unless approved. What we take extreme care at all times to ensure the final responsibility must be taken by our client. IF YOU REACH THE PLACE YOU ARE RECHERCHING FULL APPROVAL OF DESIGN AND TEXT.

Template No.: 51ramien 320x420_80x52.5 mm.indd
 Otsuka Template No.: T-3986-01
 Dimensions: 320 x 420 mm
 Template Revision No.: 1
 Template Revision Date.: 10 June 2025

Key

KEYLINE

TECHNICAL INFORMATION

PROFILE

DO NOT REMOVE FROM ARTWORK